



# Momentum 10x12 Wireless Installation Guide

---



April 2, 2018

Version 1.01

## Table of Contents

<b>1</b>	<b>SPECIAL NOTE</b> .....	<b>3</b>
<b>2</b>	<b>PACKAGING</b> .....	<b>3</b>
2.1	EXAMINE PACKAGING .....	3
2.2	VERIFY PACKAGE CONTENTS .....	3
<b>3</b>	<b>CONNECTION &amp; SETUP</b> .....	<b>4</b>
3.1	INSTALLATION DIAGRAM OVERVIEW .....	4
3.1.1	PoE Switch/Hub Connection .....	5
3.2	NETWORK CONNECTION .....	5
3.3	CALIBRATION .....	6
<b>4</b>	<b>GENERAL INFORMATION, OPERATION, MAINTENANCE</b> .....	<b>6</b>

## 1 Special Note

Please read and review the Aero-DR Operation User Manual before proceeding with this guide. The Aero-DR Operation User Manual is included with the unit, and can be found on the 20/20 Imaging website. By proceeding, you agree that you have read and understand all content within the User Manual.

## 2 Packaging

### 2.1 Examine packaging

Examine the packaging for any damage. Notify the 20/20 Imaging sales coordinator/manager immediately if any damage to the boxes has occurred.

### 2.2 Verify package contents


1. Detector Panel



2. Enclosure



3. Battery Charging Base & Power Cable

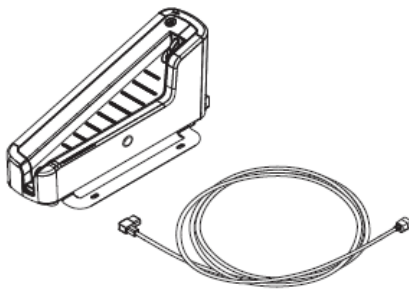

4. Access Point & Table Stand



5. Ethernet Hub



6. Ethernet Cables



7. CD-ROM with Calibration Data & User Manual




## 3 Connection & Setup

### 3.1 Installation Diagram Overview


Connect and power on all components together as shown in the following diagram.

**\*DO NOT** connect office internet to internal LAN or provided network switch. Connect office network separately using a USB to Ethernet adapter.

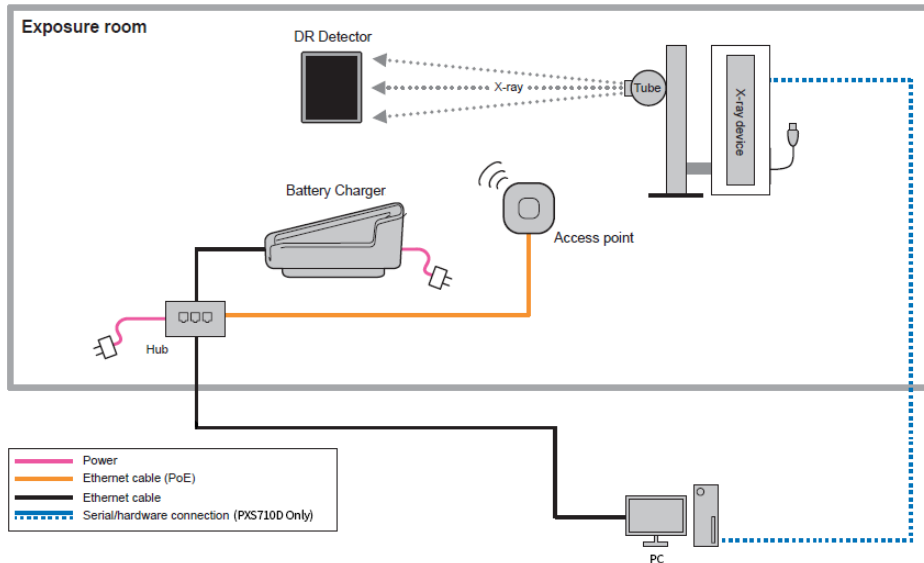


### 3.1.1 PoE Switch/Hub Connection

Ensure the Access Point is connected to a 'PoE' port so that it can receive power.

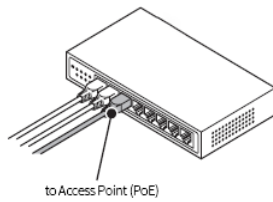


## 3.2 Network Connection

1. Open Local Network Management and click on the Momentum-DR connection.

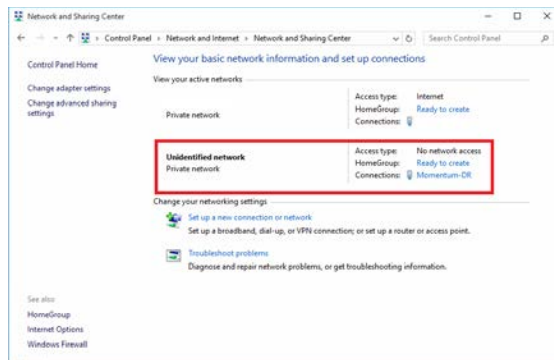

2. Open TCP/IPv4 properties in the dialogue box, highlight TCP/IPv4 and click **Properties**.

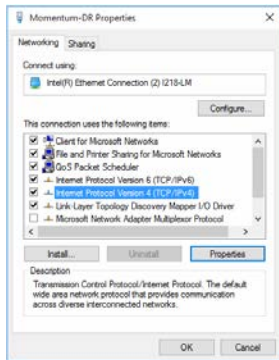

3. Ensure the IP address and Subnet Mask are set (192.168.20.100 / 255.255.255.0).



### 3.3 Calibration

Refer to the Momentum-DR Calibration Guide to set calibration data and/or perform calibration.

## 4 General Information, Operation, Maintenance

Please refer to the Aero/Momentum 1012 User Manual. The User Manual is included with the unit and can be found on the 20/20 Imaging website.

If you encounter issues not addressed by this user guide, please refer to the current Aero-DR Operation/User Manual or contact technical support for additional assistance.