

# How to Setup and Use Modality Worklist (MWL)

## Setup Steps

1. Open Opal-RAD Local System Configuration by selecting: Start\Programs\Opal-RAD StudyList\Local System Configuration.
2. Select the “MWL Settings” tab on the left side of the window.
3. Fill out the fields:
  - **My AETitle**, is the AE title of the work list you are verifying.
  - **AETitle** is that of the MWL Server.
  - **Address** is the IP of the MWL Server.
  - **Port** is that used to connect to the MWL Server.
  - **Station Name** is a customized field.
  - **Query AE Title** is the modality of what you are verifying.
  - **Status, Institution, Modality** and **Date Range** are optional fields.
4. Select “Save”
5. Open Opal-RAD Client.
6. Click on “Advanced” from the top menu and then click on “Configuration”.
7. Press the “Advanced” button and type “1234567890” and click on “OK”.
8. Select your type of Acquisition that is connected to this station and click on “Finished”.
9. Select the Modality Work list tab “MWL”. Figure 2



Field	Value
My AETitle	OPAL_CR
AETitle	SCHEDULE
Address	192.168.5.44
Port	4104
Station Name	MyPC
Query AE Title	XRAY
Status	
Institution	
Modality	
Date Range	2

Figure 1

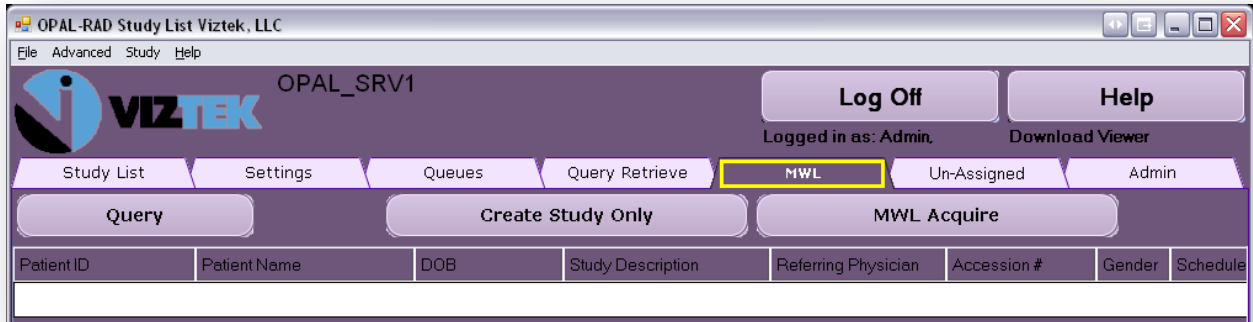


Figure 2

10. Select the “Query” and you will see the work list for the modality you entered into the Local System Configuration. Figure 3
11. Proceed to acquire Images for a listed study, via highlighting a study and then clicking on then “MWL Acquire” button.

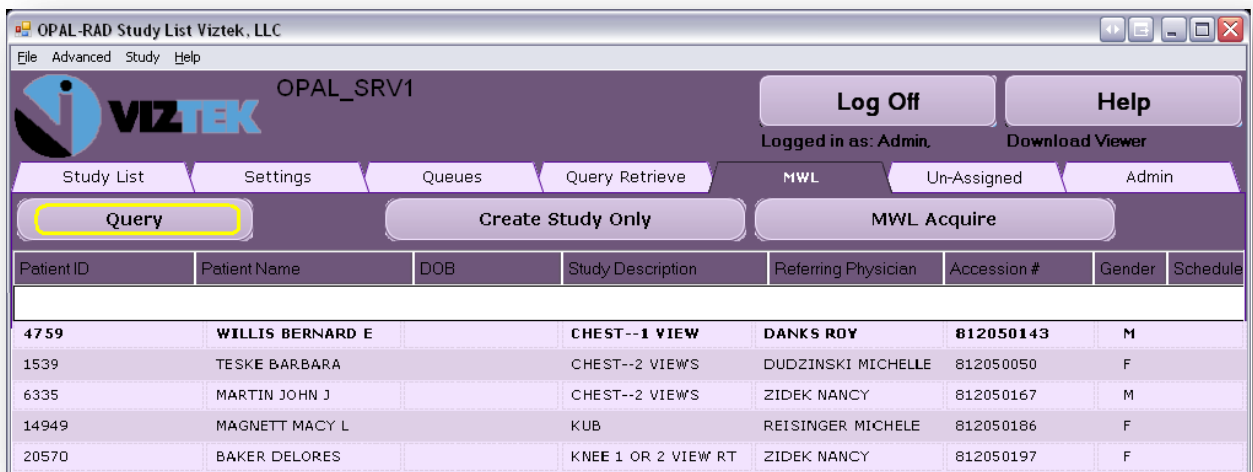


Figure 3

\*\*\* Be aware that in order to have MWL functional or display as a tab, your server has to have this optional module enable \*\*\*