

How to Edit a Study

To Edit a Study:

1. Right Click on the Patient's Name to highlight the study.
2. Right click *Edit Figure 1*
3. Select *Edit Study*
4. Edit the required field(s) *Figure 2*
5. Right click *Save*
6. Once *Updated Successfully Figure 3*
7. Select *Close*

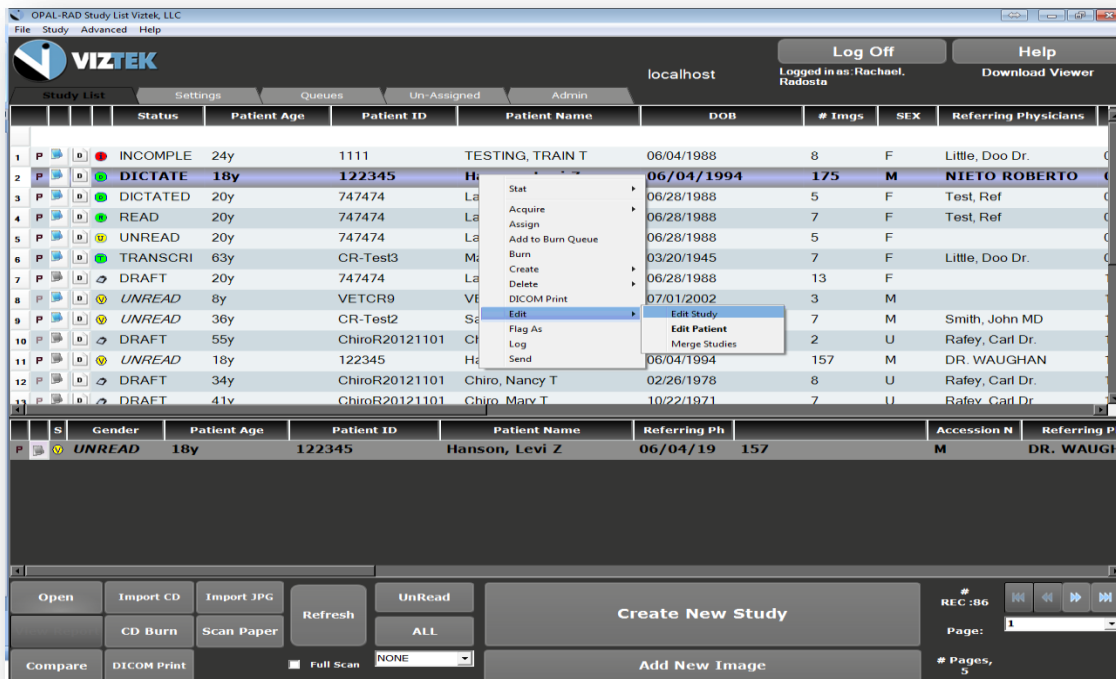


Figure 1

The screenshot shows the VIZTEK OPAL-RAD Study List application interface. The window title is "OPAL-RAD Study List Viztek, LLC". The menu bar includes "File", "Study", "Advanced", and "Help". The VIZTEK logo is in the top left. The user is logged in as "Rachael Radosta" on "localhost". The interface is divided into several sections:

- Navigation:** "Study List", "Settings", "Queues", "Un-Assigned", "Admin".
- Form Fields:**
 - Patient ID:** 122345
 - First Name:** Levi
 - Last Name:** Hanson
 - M.I.:** Z
 - Gender:** Male (selected), Female, U/K
 - D.O.B.:** 06/04/1994
 - Study UID:** 1.2.840.113619.2.182.120254181220113241.1351259169.185452
 - Accession #:** 1001442045
 - Study ID:** 18589
 - Study Date/Time:** 10/26/2012 11:02
 - Modality:** MR
 - Flag:** None
 - Stat:** None
- Study Description:** MRI HEAD/BRAIN W/O CONTRAST
- Body Part:** <Select Body Part>
- Reading Physician:** Radosta^Rachael^L^^
- Referring Physicians - NIETO ROBERTO M^^^^:** <Select Referring Physicians>
- Institution Name:** MEDICAL CENTER OF ARLINGTON
- Facility:** <Select Facility>
- Department:** <Select Department>

At the bottom right, there are three buttons: "Toggle Edit Patient", "SAVE", and "CLOSE".

Figure 2

This image shows a close-up of the bottom right portion of the application window. It features a dark grey background with a white border. At the top left, the text "Update Success" is displayed in a bright green font. Below this, there are three stacked buttons with a light grey gradient and rounded corners:

- The top button is labeled "Toggle Edit Patient". A white mouse cursor icon is pointing at the left side of this button.
- The middle button is labeled "SAVE".
- The bottom button is labeled "CLOSE".

Figure 3