

# Mammography Hanging Protocols

---

This guide outlines the 7 steps to create and save the hanging protocols for Mammography.

 *Rule of thumb to remember ... Split- Drag- Save*

## 7 Steps to Hanging Protocols

◆ **Prerequisite:** “Use Custom Mammography Layout” must be activated.

I. For directions please review page 5.

1. To launch the Opal Viewer
  - a. Select a patient with a MG study from the Opal worklist and left click open  
OR double left click a patient to launch the Opal Viewer

2. Select a matrix layout from the Viewer toolbar. (Figure 1)



Figure 1

3. Select Image to drag into the Opal Viewer Image Window (Figures 4-5)

4. Select the save button (Figures 2-3)


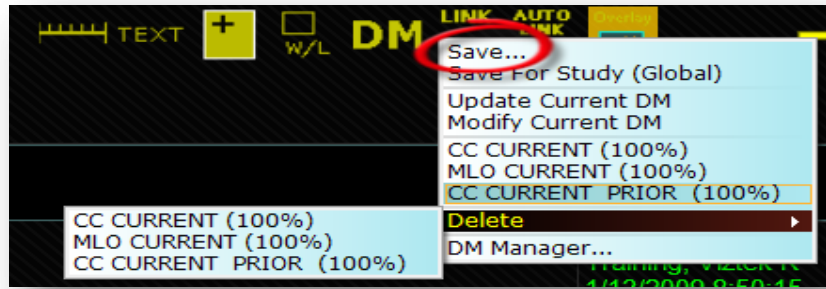


Figure 2

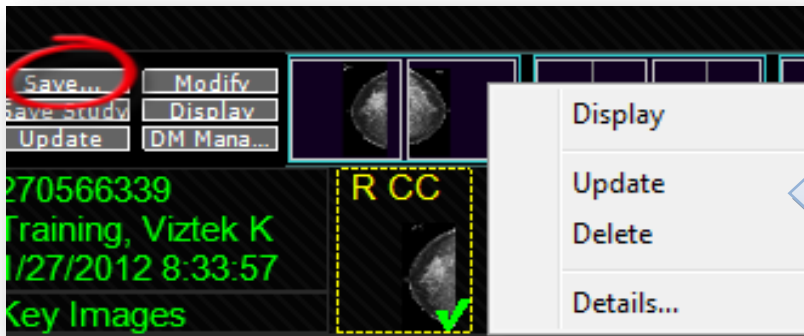


Highlight protocol set to activate drop down then right click.

**Display:** will display the set highlighted (same as double click)

**Delete:** will delete the set

Figure 3

5. Create a naming label for set that was created for the hanging protocol. (Figures 4)

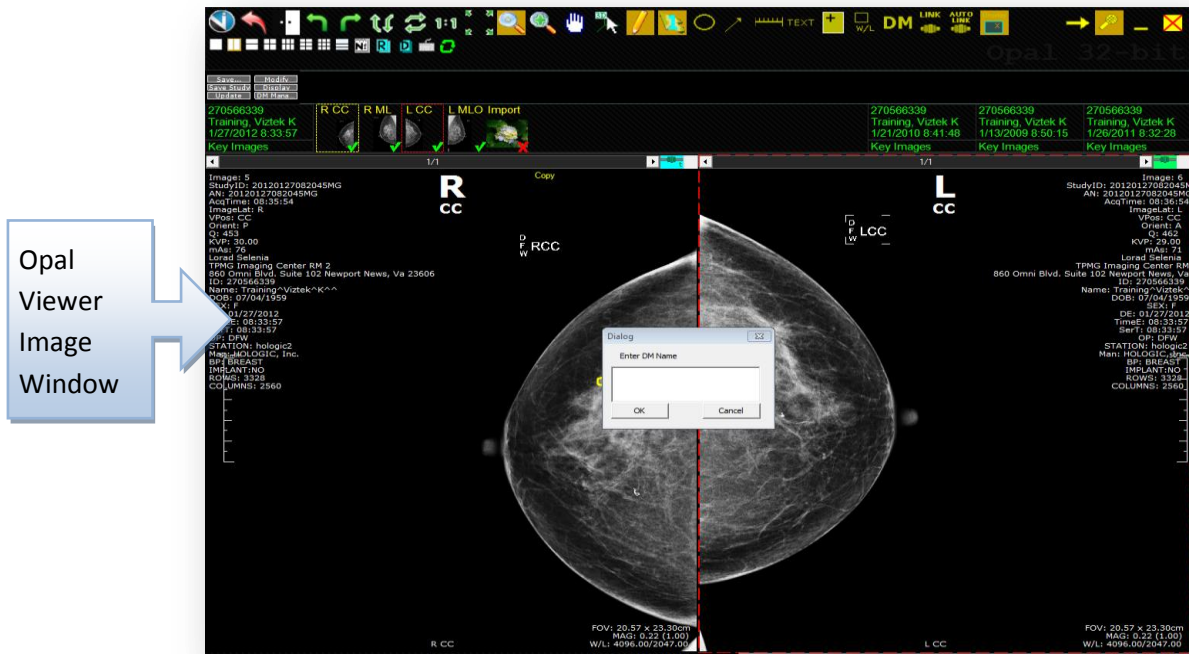

Opal  
Viewer  
Image  
Window

Figure 4

6. Use the [ - ] keys to cycle through the DM layouts to view hanging protocol sets

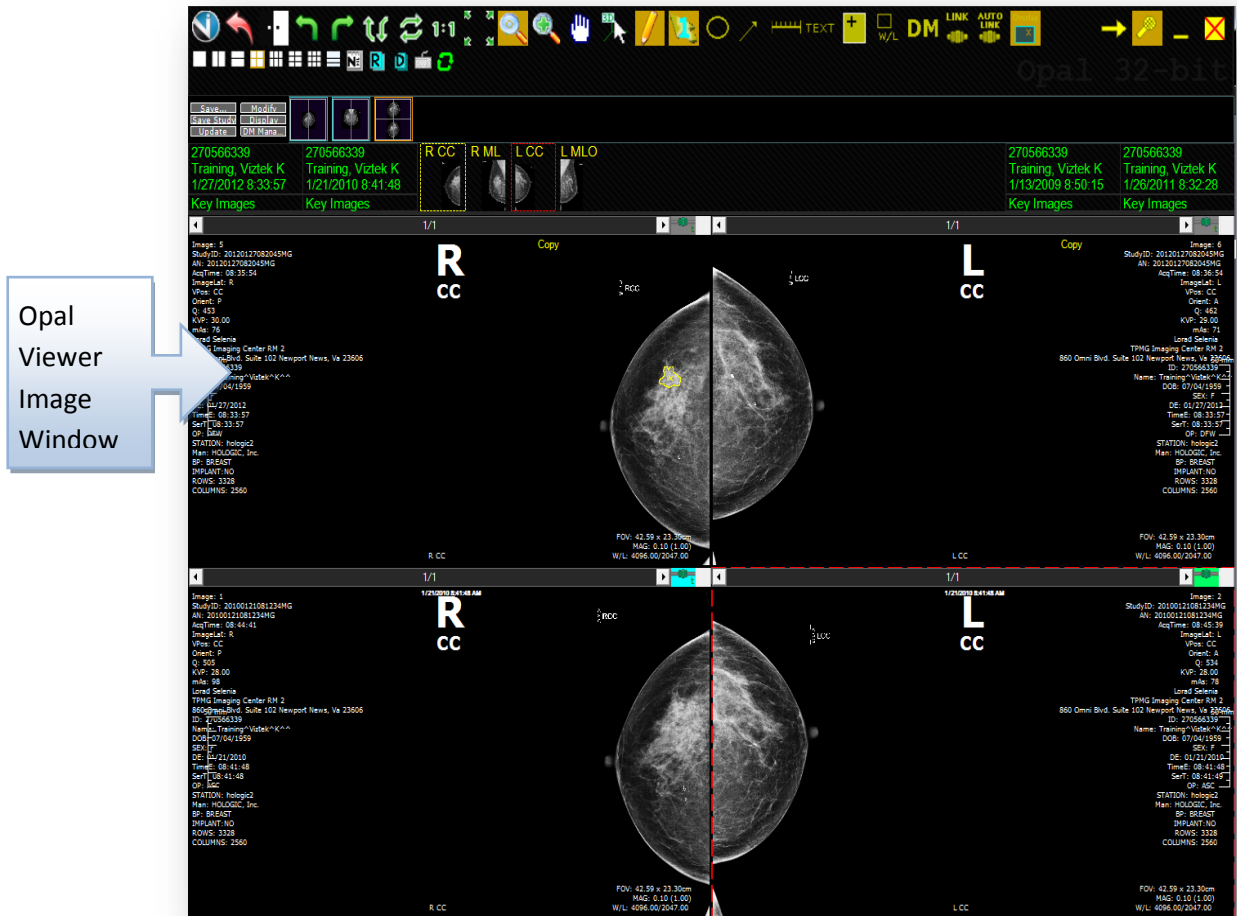


Figure 5

- Please note: Figure 4-5 settings are based on using the option “Use 2+ years old prior” as well as having the option “Show DM Tool” activated and using a *single monitor*.

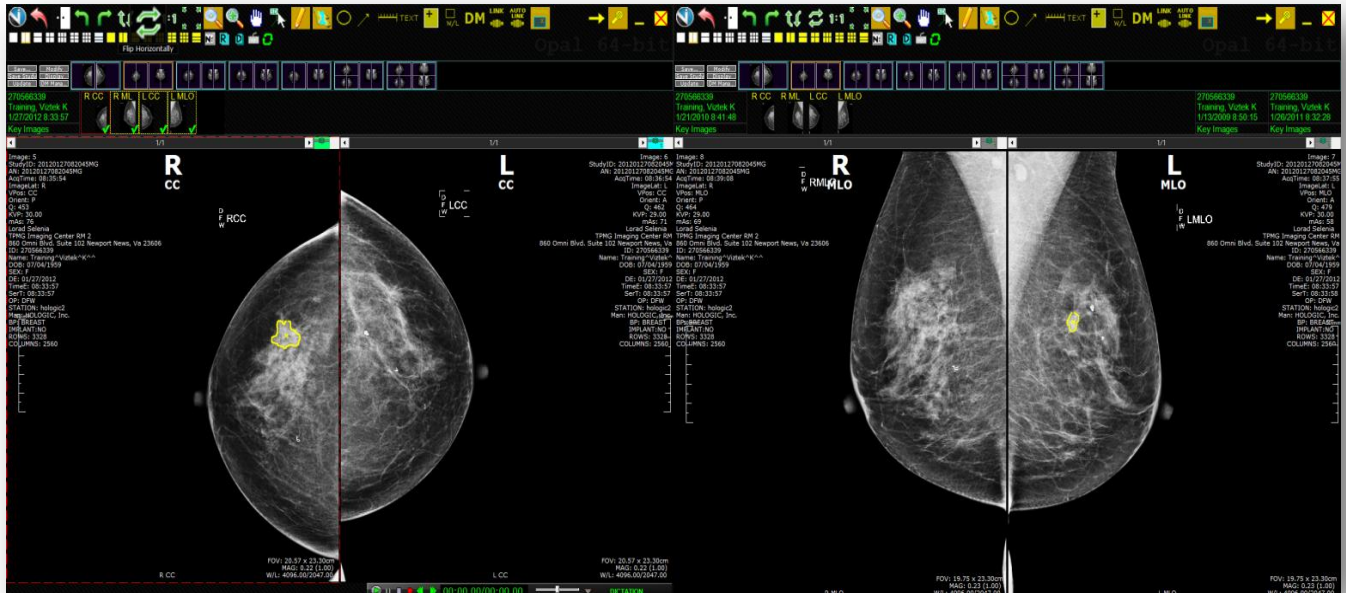


Figure 6

- Please note: Figure 6 settings are based on using the option “Use 2+ years old prior” as well as having the option “Show DM Tool” activated and using a *dual monitor*

7. Modify hanging protocol if needed.

- Select the DM Manger from either the DM Manager Toolbar (Figure 7) or by selecting the DM on the toolbar(Figure 8)

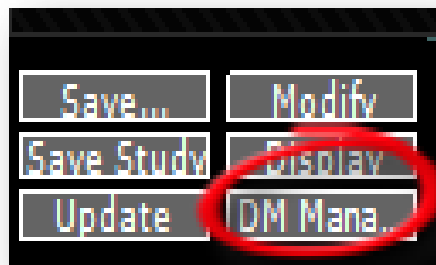


Figure 7

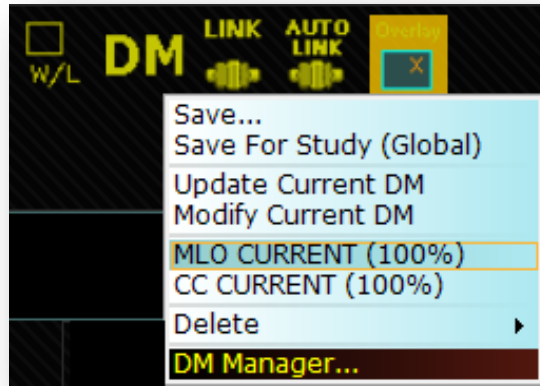


Figure 8

- b. Highlight protocol set, right click to activate drop down menu.
  - i. **Display:** will display the set highlighted (same as double click)
  - ii. **Delete:** will delete the set
  - iii. **Detail:** allows name to be edited for the hanging protocol set

## Prerequisite

- To activate - Use Custom Mammography Layout

1. Enter the Opal Image Viewer Module by double clicking any study from the worklist. From here, you have the ability to change a variety of settings through the user SETTINGS option.
2. Click the Viztek Icon to activate the main menu.
  - Click Settings >> Edit
  - Select the Mammography Options Tab
  - Select the option *Use Custom Mammography Layout*
  - Left click Save.

