


## Prevent Opal Web from Automatically Opening in Microsoft Edge

Microsoft Edge is a new browser from Microsoft that automatically opens any site you try to open in Internet Explorer. This can be disabled via a setting in the Edge browser.


Step 1:  
Open Microsoft Edge.




Step 2:  
Click on the 3 dots in the top right corner and click “Settings” from the drop-down list.

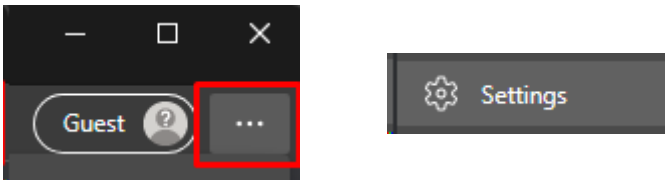

Step 3:  
On the left side of the settings page, click “Default Browser”.

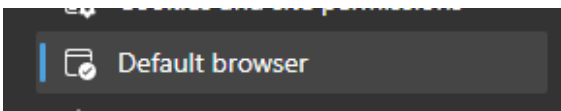

Step 4:  
Under “Internet Explorer compatibility” click the drop-down list and change the setting to “Never”.

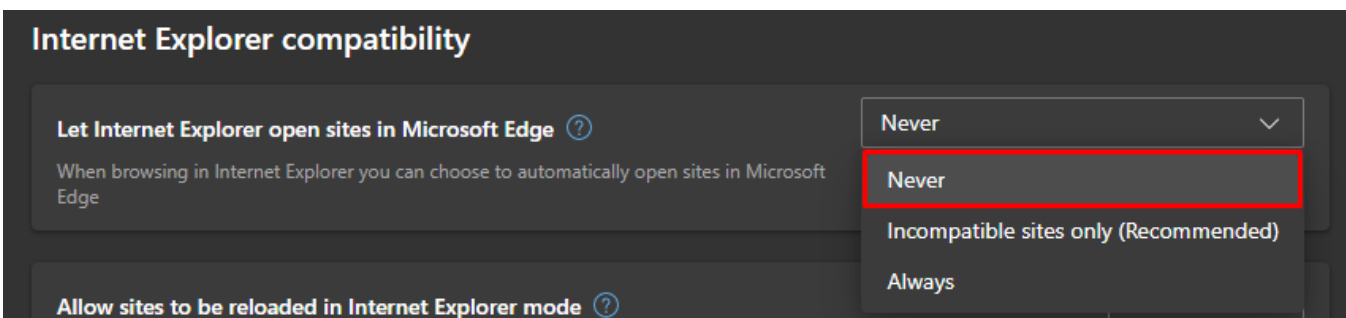

Opal Web will now work in Internet Explorer once more.