

Setup Opal-RAD Workstation for viewing - Internet Explorer 11+

NOTE: Internet Explorer must be used, all other web browsers are not compatible for full viewing*

1. Open Internet Explorer Browser

NOTE: If using Windows 8/8.1, ensure you open Internet Explorer from the 'Desktop' and not the 'Start' (tile) page

NOTE: If using Windows 10, ensure you are using Internet Explorer, and not the 'Edge' browser

2. Type the location of the Server in the Address bar:

(ie. http://OPAL_QC/opalweb/login.aspx or <http://IP Address/opalweb/login.aspx>) and hit Enter

NOTE: You may see the mOpal login screen, this is normal and will be corrected below

NOTE: If you receive a prompt, select 'Turn on Intranet Settings'




3. Enable 'Compatibility View'

1. Look for the Tools button . It will appear in top right side of the browser window

NOTE: if you do not see this, use 'Alt' key and select 'Tools' dropdown near the top, and continue below

2. Click the button and select 'Compatibility View settings' option

3. Click the 'Add' button

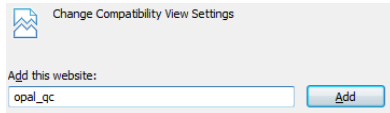

4. Click 'Close'


4. Restore down the browser window (so it is not full screen and you can see some of the desktop)

5. Click and drag the 2020 logo/icon to left of the address bar onto the Desktop to create a shortcut



6. Login to Opal-RAD at the login screen

7. Viewer: Click 'Download Viewer - 2.x.x.x' near the top right, and allow it to install

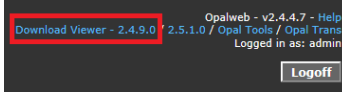


NOTE: It may automatically prompt to install the viewer, which you can use this method skipped the above step



Click 'Run' near the bottom of the screen and allow the viewer to install

Tools (CD Burning & Import): Click 'Opal Tools' near the top right, and allow it to install

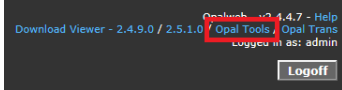

Click 'Run' near the bottom of the screen and allow the viewer to install

NOTE: After OpalViewer / Opal Tools are installed, you may be asked to allow 'OpalWebKicker'. This is necessary for viewing software to run correctly



Click 'Allow'. The session might log you off, this is normal. Simply log back in

10. Your workstation setup is now complete

*mOpal is available for other browsers as well as mobile and smart devices. See website or contact 20/20 Imaging for details