

# How to Change the Server IP Address Guide

1. Click on *Start*
2. Click *All Programs*.
3. Go to *Opal-RAD StudyList*.
4. Left click on *Local System Configuration*. Figure 1
5. Click Edit to change Server IP Address. Figure 2
  - a. Other options that could be used
    - i. Click “Add” to add another Server IP address to the menu.
    - ii. Click “Delete” to remove an already existing Server IP Address
6. Change *Server IP* to {**new Server IP Address**} i.e.: 199.168.33
7. Enter New Name
8. Click on *Update* then *Save and Exit* Figure 3

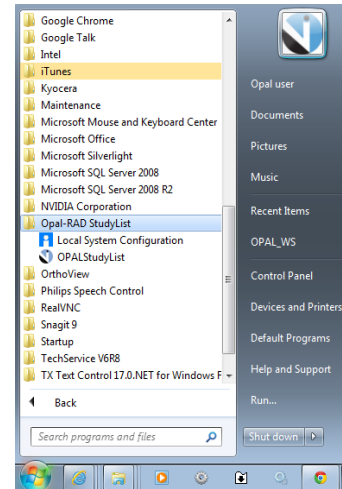


Figure 1

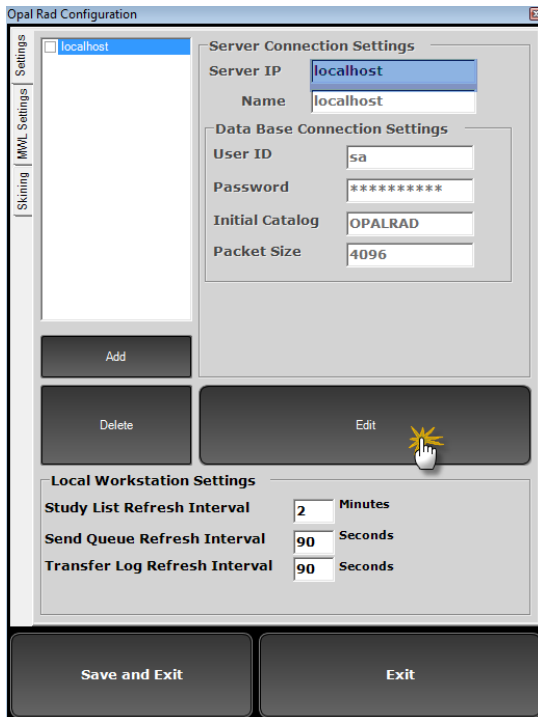


Figure 3

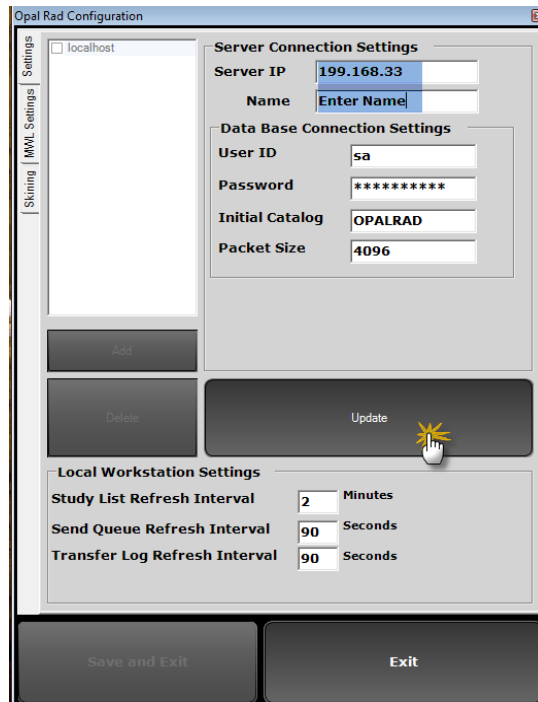


Figure 2