

PODIATRY XRAY SETTINGS* (AVERAGE) XMD Series

| | |
|---|--|
| <p>FOOT</p> <p>AP/DP Medial Oblique Lateral Oblique Lateral</p> | <p>62kv, 014 pulses 62kv, 014 pulses 62kv, 014 pulses 62kv, 018 pulses</p> |
| <p>ANKLE</p> <p>AP/DP Medial Oblique Lateral Oblique Lateral</p> | <p>62kv, 022 pulses 62kv, 018 pulses 62kv, 018 pulses 62kv, 018 pulses</p> |
| <p>KNEE**</p> <p>AP Oblique Lateral</p> | <p>65kv, 028 pulses 62kv, 024 pulses 62kv, 024 pulses</p> |
| <p>OTHER</p> <p>Calcaneal Axial Harris Beath Sesamoids</p> | <p>62kv, 014 pulses 62kv, 028 pulses 62kv, 016 pulses</p> |

SMALL (Child under 8): lower 2-4 pulses

LARGE: add 2-4 pulses

CAST: add 4 pulses

*All views to be shot at 15ma setting. *This is only a reference. kv and pulses can be adjusted when necessary.*

Calibration:

Luminance: 55kv, 020 pulses

Positional: 50kv, 012 pulses

© 20/20 Imaging LLC - All Rights Reserved. *This chart is only a guide. 20/20 Imaging is not responsible for under/over exposure , or improper patient exposure problems. Your result may vary. For assistance, please call support at 866-734-6234. **Knee views are not supported and may be taken at your own risk.