



PXS-710D

PODIATRY XRAY SETTINGS\* (AVERAGE) XM3 Series

Examination	Thickness	KVP	MAS	SID	Notes
<b>Foot</b>					
Dorso-plantar (AP)	Normal	55 -60	1.5	30"	
Lateral	Normal	55 -60	1.5	36"	
Marked Lesion	Normal	55 -60	1.5	30"	
Lateral Oblique	Normal	55 -60	1.5	36"	
Medial Oblique	Normal	55 -60	1.5	30"	
Axial Calcaneal	Normal	55 -60	4.5	30"	
Axial Sesamoid	Normal	55 -60	4.5	30"	
Harris and Beath	Normal	55 -60	4.5	30"	
Lewis	Normal	55 -60	3.0	30"	
Causton	Normal	55 -60	3.0	30"	
<b>Ankle</b>					
Anterior-Posterior	Normal	55 -60	3.0	30"	
Lateral	Normal	55 -60	3.0	36"	
Lateral Oblique	Normal	55 -60	3.0	36"	
Medial Oblique	Normal	55 -60	3.0	30"	
Mortise	Normal	55 -60	3.0	30"	
Stress Views (AP, lat)	Normal	55 -60	1.5	30"/36"	
Holly	Normal	55 -60	1.5	36"	
Isherwood	Normal	55 -60	1.5	36"	
Broden	Normal	55 -60	3.0	36"	

SMALL (Child under 8): lower MAS .5

LARGE: increase MAS .5

CAST: increase MAS 1.0

© 20/20 Imaging LLC - All Rights Reserved. \*This chart is only a guide. 20/20 Imaging is not responsible for under/over exposure , or improper patient exposure problems. Your result may vary. For assistance, please call support at 866-734-6034.

