

Opal-Rad Workflow

Basic Status Workflow:

	INCOMPLETE	Images being received by modality and have not yet been validated.
	UNREAD	Images received by modality and validated by technologist or server.
	UNREAD	Images have been viewed
	READ	Images viewed, no dictation performed, study closed, study marked read.

Additional Statuses – Dictation/Transcription sites only:

	DICTATED	Images viewed, dictation performed, study closed, dictation uploaded.
	DRAFT	Transcription performed, "Save Transcription" button used.
	TRANSCRIBED	Transcription performed, "Submit for Review" button used.
	NOT APPROVED	Radiologist rejected preliminary report.
	APPROVED	Radiologist approved preliminary report. Report is saved.