



KONICA MINOLTA

Symmetry Mini PACS & Symmetry Sync

Version 1.0

Workstation Setup Guide

500-000858B



Revision History

Revision	Date	Reason for Change
A	March 2025	Initial release
B	November 2025	Updated document title, added steps to update shortcut properties, removed invalid SSL, and added Google Chrome cache clear



Distributed by:

Konica Minolta Healthcare Americas, Inc.

2217 US Highway 70 East


Garner, NC 27529

1-800-366-5343


Setting up a Symmetry Sync Workstation

To set up a workstation, complete the following steps:


1. Open Google Chrome web browser.
2. Browse to **http://SYMSYNC** (or static IP address if available).
3. The Symmetry Sync Login screen opens.

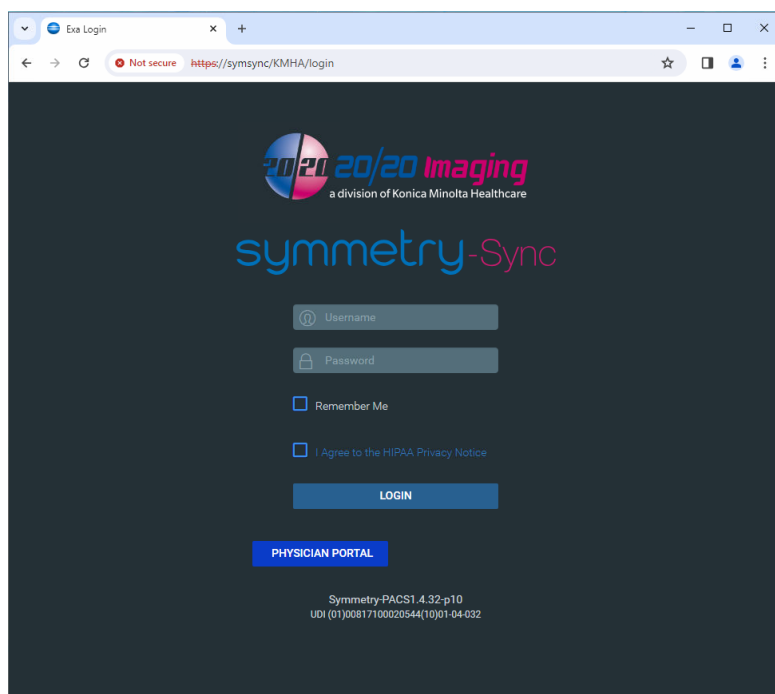


Note: Confirm the window is **not** maximized. If it is, click the **Restore** button at the top-right corner of the window.

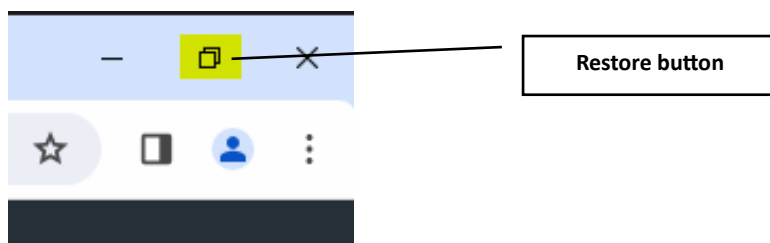

- Click the **Menu** icon in the top-right corner of the Chrome window. Then click **Save and share > Create Shortcut**.

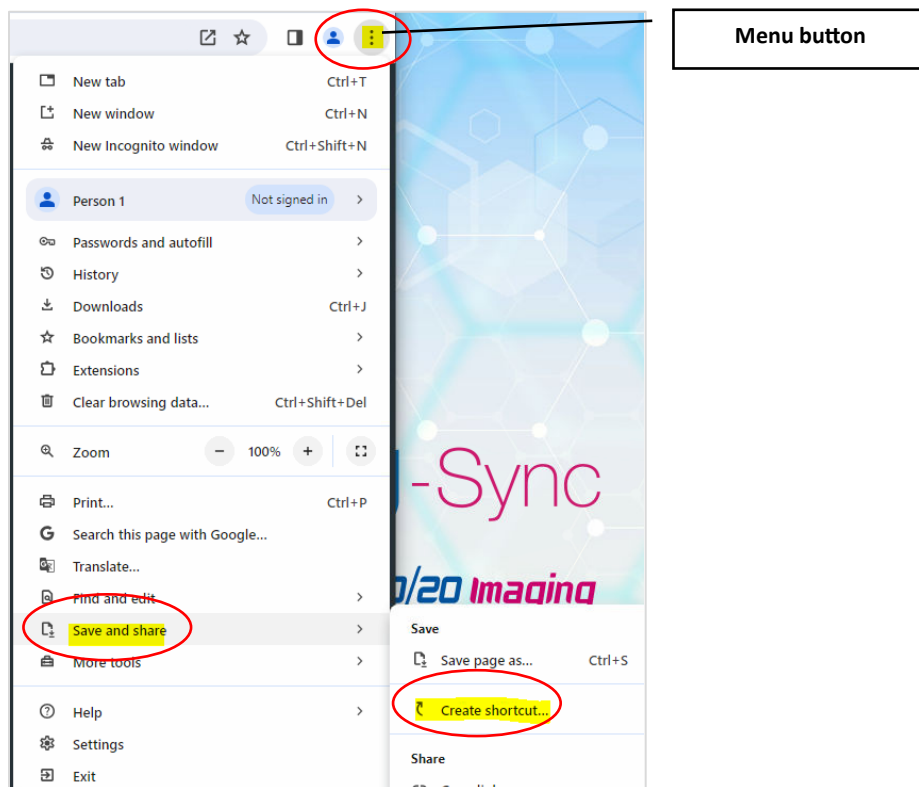


- In the Create Shortcut box, type **SymSync Login** in the text field and click **Create**.

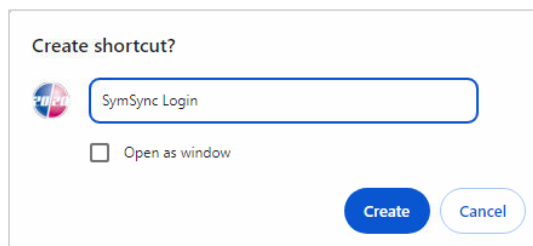


NOTE: Do **not** check the **Open as window** box.

- Confirm that the shortcut is available on the desktop.

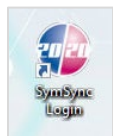

- Launch Symmetry Sync from the shortcut.


8. In the Symmetry Sync Login screen:
 - a. Type your user credentials.
 - b. Check the **I agree to the HIPAA Privacy Notice** box.
 - c. Click **Login**.
9. Click the **Menu** icon in the top-right corner of the Chrome window, then click **Settings> Privacy and Security**.
10. On the Basic tab, confirm that the Time Range drop-down box is set to **All time**.
11. Uncheck all options except the **Cached images and file** box.

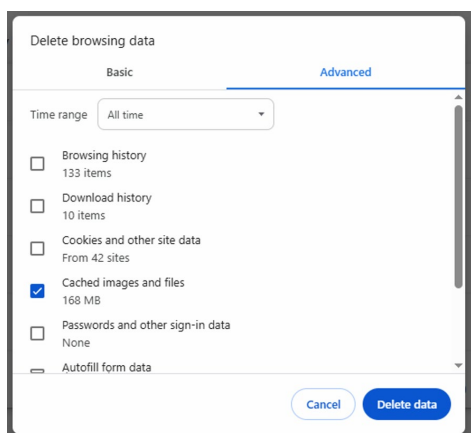


12. Click **Delete data**.

Note: This clears any old OpalViewer and OpalTools downloads that get cached by Chrome and prevent older version from being reinstalled.

13. Click the **hamburger** button in the top-left corner of the Symmetry Sync screen.

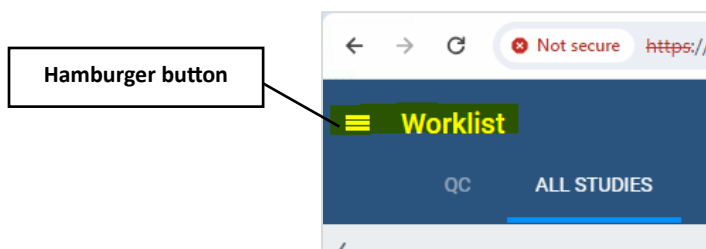


14. Select **Tools> Opal Viewer**.

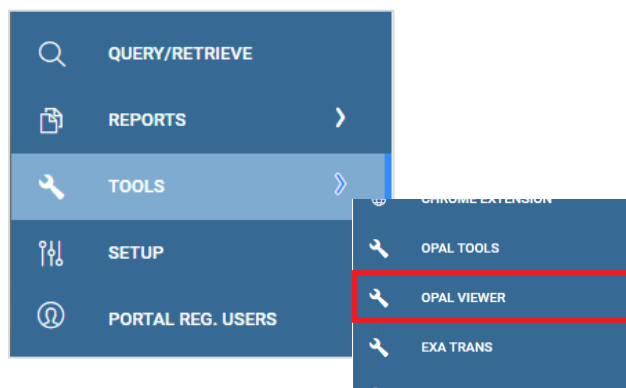


Note: ExaClientViewerSETUP.exe will be downloaded to the default Downloads folder.

15. Run the ExaClientViewerSETUP.exe file.

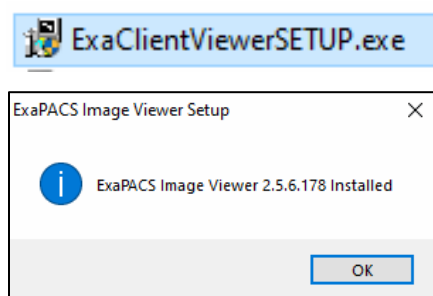


16. Select **Tools> Opal Tools**.

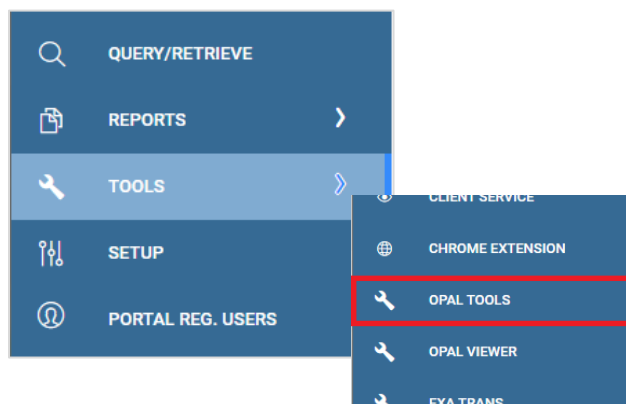


OpalToolsSETUP.exe will be downloaded to the default Downloads folder.


- Run the OpalToolsSETUP.exe file.

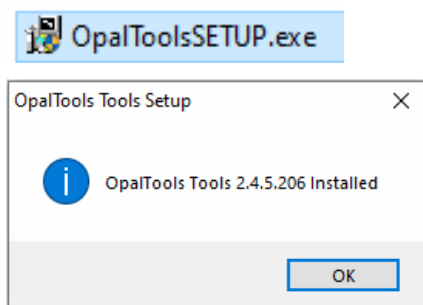


- Click the **Gear** icon next to the Power icon.

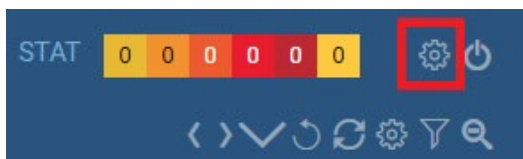

- Select **User Settings** from the drop-down menu.

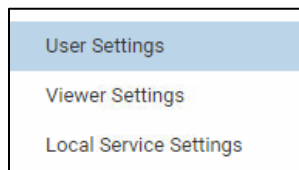

- Click the **Double-Click Behavior** drop-down arrow and select **Opal Viewer**.

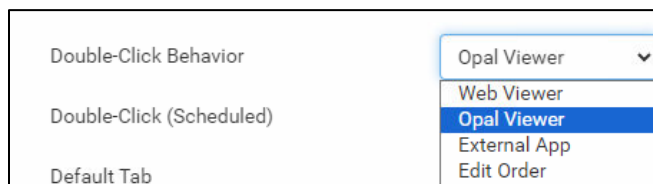

- Click **Save** and close the popup window. Your Symmetry/Symmetry Sync workstation setup is complete.