



ChiroTouch Integration Use Guide

OpalViewerSETUP-2.5.4.19 or above required (ie 2.5.5.x+)

- * Contact your x-ray dealer or 2020 Imaging for setup/pre-configuration prior to use

Opal-RAD

1. Simply open the study via the Opal Studylist at the configured PC (to open the Opal-Viewer software), wait until all images are loaded/shown
2. Ensure you view EVERY image so each x-ray view is exported, and then close the study when finished
The JPEG files will automatically be created to the specified/configured directory
*PLEASE NOTE: Images may not include saved annotations

ChiroTouch

1. Click 'New', then 'File'



2. Select 'Import File' button



3. Shift or Ctrl select the JPEG images and select Opal
NOTE: Images are set to a particular format; 'PatientID – StudyDate – View'
Images are now available via the patient's chart