

# HOTKEYS CHART

Path to Access: Opal Viewer Main Menu > Help > Hotkeys Chart

General Viewer		
Key	Function	Description
O	Overlays	Switches between display of DICOM tag values.
I	Image Map	Shows/hides on-screen thumbnail of image and view rectangle.
C	Cutlines	Toggles display of cutlines.
SHIFT+C	SR CAD	Toggles display of SR CAD.
A	Annotations	Toggles display of annotations.
R	Red/Blue Swap	Swap Red and Blue color channels (for color images only).
T	Actual Size	Set zoom level to display image at true size (monitor must be calibrated).
F	Fit to Window	Set zoom level to fit the image in available display.
V	DICOM Overlay	Toggles display of modality-provided DICOM overlay (if available).
K	Key Image	Toggles the current image as a key image.
E	Ellipse Annotation	Activates Ellipse Annotation
M	Measure Annotation	Activates Measure Annotation
CTRL+0	Auto W/L	Uses an algorithm to determine an appropriate window and level setting.
CTRL+ALT+0	Auto W/L (Series)	Uses a series algorithm to determine an appropriate window and level setting.
0	Reset W/L	Resets window and level values to last saved.
1-9	W/L Presets	Selects one of 9 available preset window and level values.
CTRL+1-9	Adtl. W/L Presets	Selects one of 9 additionally available preset window and level values.
L	Auto Link	Links or unlinks all possible images.
CTRL+L	Show DM Toolbar	Show/Hide DM Toolbar
< and >	Traverse Image	Steps through the image as zoomed-in pieces either forwards/backwards.
[ and ]	Cycle DMs	Cycles through applicable display layouts (DM).
Up and Down	Scroll Images	Displays the previous or next image in the current cell.
PgUp and PgDown	Scroll Series	Displays the previous or next series in the current cell.
CTRL+N	Auto Open Next	Toggle Auto Open Next.
CTRL+D	Auto Upload Dictation	Toggle Auto Upload Dictation.
CTRL+M	Auto Mark Read	Toggle Auto Mark Read.
F12	Premium View	Applies Premium View Bone Enhancement if set.
F5	Fullscreen	Toggles hiding the toolbars to display image in fullscreen.
F1	Settings Manual	Opens Settings Manual.