

## Setup Opal-RAD Workstation for viewing *(browser based)*

1. Open Internet Explorer Browser
2. Type the location of the Server in the Address bar: (ie. [http://OPAL\\_QC/](http://OPAL_QC/) or <http://IP Address/>) and hit Enter

**NOTE: Internet Explorer must be used**, all other web browsers are not compatible\*

If desired, you may turn on/allow Intranet Settings If you receive a prompt



You may skip step 3 if Intranet settings are enabled (*enables for Server connection only*)


Intranet settings are turned off by default.

Don't show this message again


Turn on Intranet settings

x

3. Enable 'Compatibility View' (if available, see below)
  1. See if the Compatibility View button  appears in the Address bar. (If you don't see the button, there's no need to turn on Compatibility View.)
  2. Tap or click the Compatibility View button  to display the site in Compatibility View
4. Restore down browser (so it is not full screen)
5. Select 20/20 logo/icon to the left of the address bar, click and drag it onto the Desktop to create a shortcut

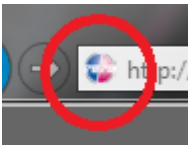

6. Login
7. You will be prompted to install '**OpalViewer**'. This is a safe program. Click OK to allow install

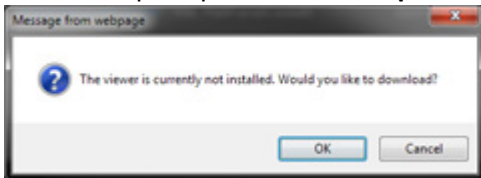

8. After OpalViewer is installed, you will be asked to allow '**OpalWebKicker**'. This is necessary for viewing software to run correctly

This webpage wants to run the following add-on: 'OpalWebKicker' from 'Viztek LLC (unverified publisher)'. [What's the risk?](#)

Allow

x

Click 'Allow'. The session might log you off, this is normal. Simply log back in

9. Your viewer is now complete

\*mOpal is available for other browsers as well as mobile and smart devices. See website or contact 20/20 Imaging for details