



## Setup Opal-RAD Workstation for viewing – Microsoft Edge 97.x+


1. Open Microsoft Edge Browser

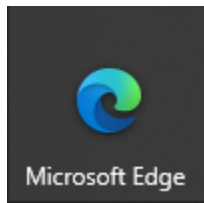


2. Enable 'Internet Explorer mode'  
Open edge main Menu, (three dots on far top right)

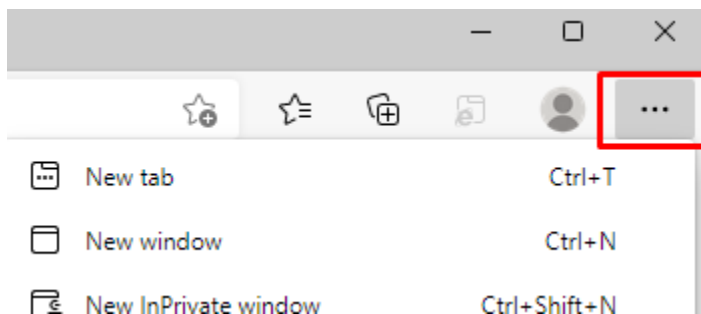

Select settings

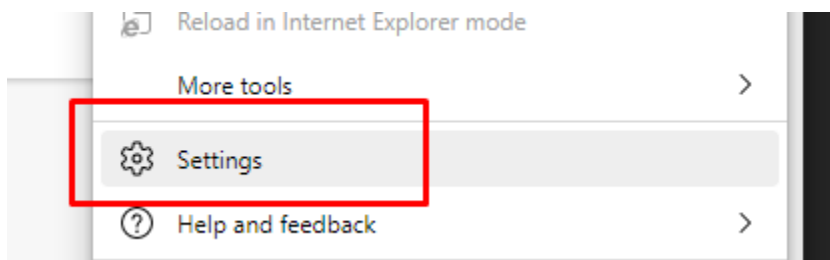


Select 'Default Browser' on the left

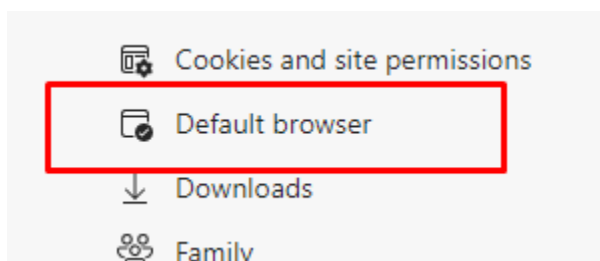


## Setup Opal-RAD Workstation for viewing – Microsoft Edge 97.x+

Select “ALLOW” from drop down ‘Allow sites to be reloaded in Internet Explorer mode’

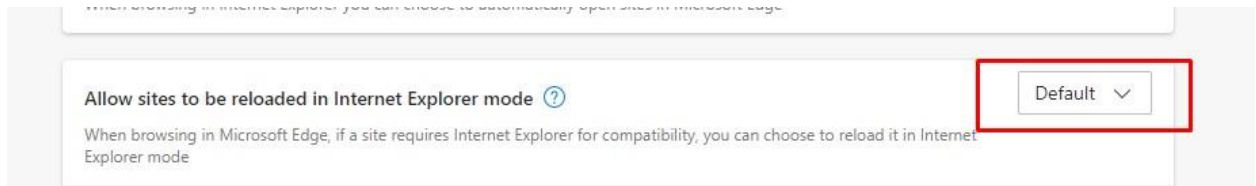

Select RESTART as promoted in the ‘Allow sites to be reloaded in Internet Explorer mode’

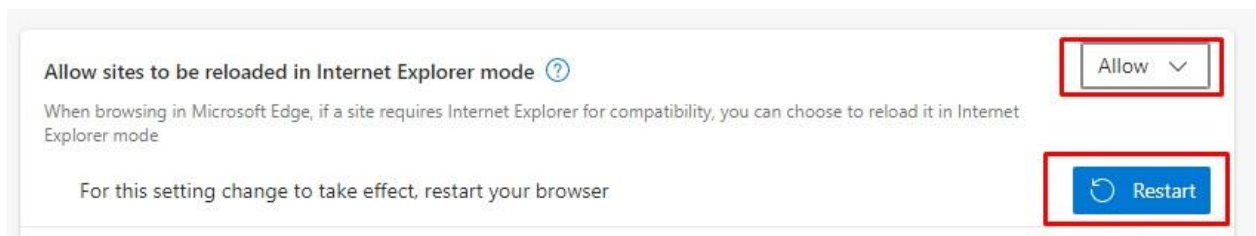


### 3. Add website to 'Internet Explorer mode’

Type the location of the Server in the Address bar:

(ie. [http://OPAL\\_QC/opalweb/login.aspx](http://OPAL_QC/opalweb/login.aspx) or <http://IP Address/opalweb/login.aspx>) and hit Enter

Select “Reload tab in Internet Explorer mode” icon

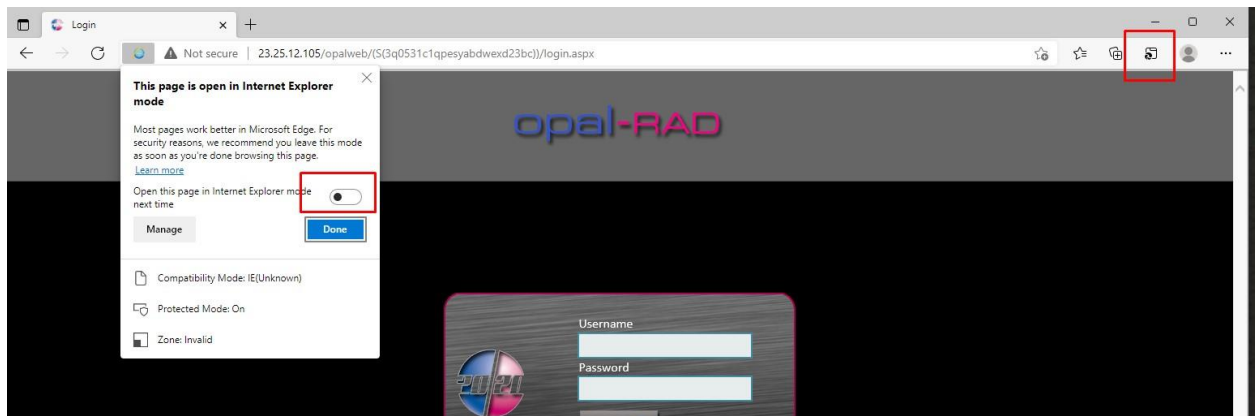


Select open this page in internet Explorer mode next time switch

Click on done.

## Setup Opal-RAD Workstation for viewing – Microsoft Edge 97.x+

### 4. Add shortcut on desktop


- Open the Edge main Menu, (three dots on far top right)
- Hover on the "Apps" menu option.
- Click on the pop-up option to "install this site as a web app".

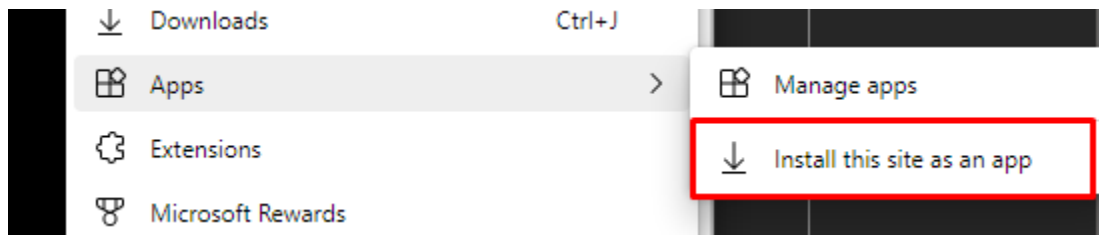

### d) click on install




### e) Tick the boxes for pinning the app to desktop, and, or start menu, (as icons). and click "Pin".

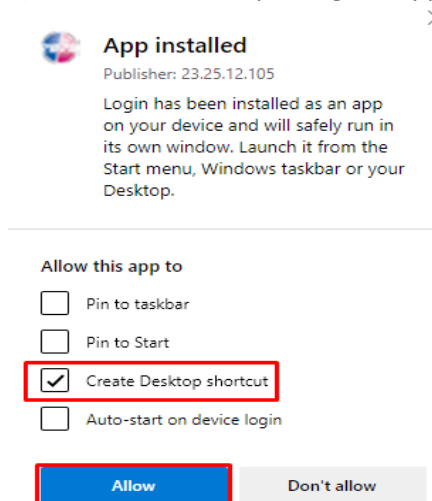


## Setup Opal-RAD Workstation for viewing – Microsoft Edge 97.x+

5. Login to Opal-RAD at the login screen

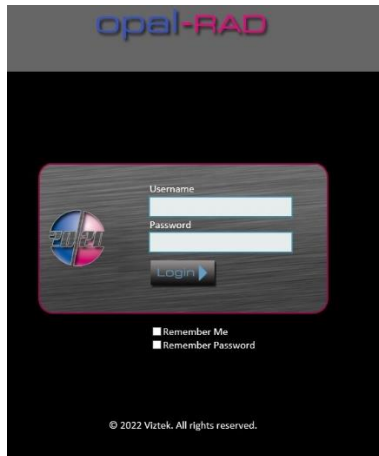

6. Install Viewer: Click 'Download Viewer – 2.x.x.x' near the top right, and allow it to install

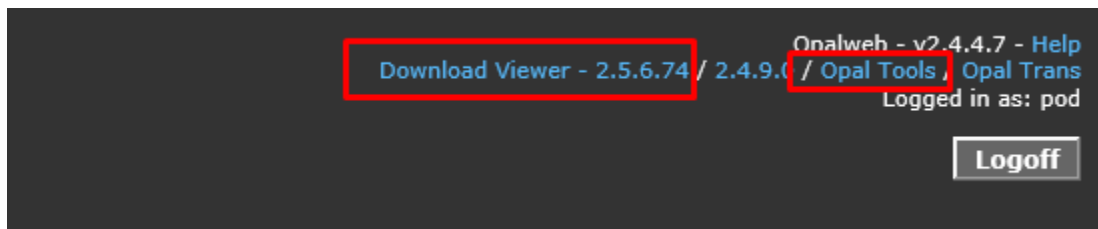

NOTE: It may automatically prompt to install the viewer, which you can use this method skipped the above step

Click 'Run' near the bottom of the screen and allow the viewer to install

7. Install Tools: (CD Burning & Import): Click 'Opal Tools' near the top right, and allow it to install


Click 'Run' near the bottom of the screen and allow the viewer to install

NOTE: After OpalViewer/Opal Tools are installed, you may be asked to allow 'OpalWebKicker'. This is necessary for viewing software to run correctly

Click 'Allow'. The session might log you off, this is normal. Simply log back in

## Setup Opal-RAD Workstation for viewing – Microsoft Edge 97.x+

- Please note Microsoft Edge only will allow 30 days for a site to load in INTERNET EXPLORER MODE currently. After the 30 days expire, you will need to add the site again via step 3 or you manually add in the Default web Browser settings.

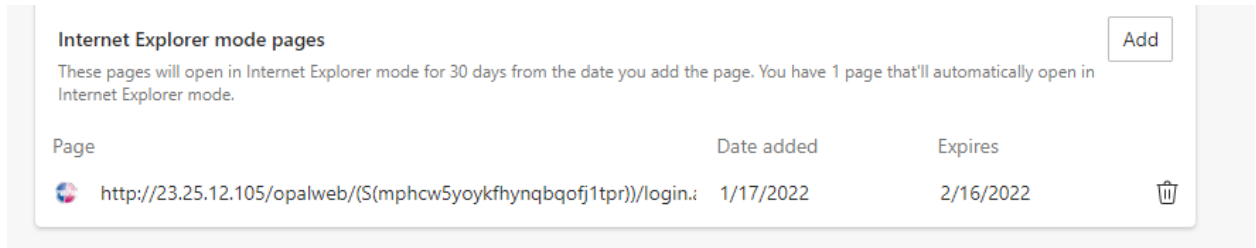

- Your workstation setup is now complete

If you encounter any issues not addressed by this user guide, please contact technical support for additional assistance.