

PODIATRY XRAY SETTINGS* (AVERAGE) LT Series

| | |
|---|--|
| <p>FOOT</p> <p>AP/DP Medial Oblique Lateral Oblique Lateral</p> | <p>65kv, 034 pulses 65kv, 036 pulses 65kv, 036 pulses 65kv, 040 pulses</p> |
| <p>ANKLE</p> <p>AP/DP Medial Oblique Lateral Oblique Lateral</p> | <p>68kv, 044 pulses 65kv, 036 pulses 65kv, 036 pulses 65kv, 036 pulses</p> |
| <p>KNEE**</p> <p>AP Oblique Lateral</p> | <p>70kv, 048 pulses 65kv, 040 pulses 65kv, 040 pulses</p> |
| <p>OTHER</p> <p>Calcaneal Axial Harris Beath Sesamoids</p> | <p>65kv, 030 pulses 65kv, 044 pulses 65kv, 036 pulses</p> |

SMALL (Child under 8): lower 4-8 pulses

LARGE: add 4-8 pulses

CAST: add 10 pulses

*All views to be shot at 15ma setting. *This is only a reference. kv and pulses can be adjusted when necessary.*

Calibration:

Luminance: 65kv, 036 pulses

Positional: 65kv, 032 pulses

© 20/20 Imaging LLC - All Rights Reserved. *This chart is only a guide. 20/20 Imaging is not responsible for under/over exposure , or improper patient exposure problems. Your result may vary. For assistance, please call support at 866-734-6534. **Knee views are not supported and may be taken at your own risk.