

PODIATRY XRAY SETTINGS (AVERAGE) for use with X-Cel X-Ray Generator Only - XM3 Series*

<p>FOOT</p> <p>AP/DP Medial Oblique Lateral Oblique Lateral</p>	<p>60kv, 008 pulses 60kv, 008 pulses 60kv, 008 pulses 60kv, 012 pulses</p>
<p>ANKLE</p> <p>AP/DP Medial Oblique Lateral Oblique Lateral</p>	<p>60kv, 018 pulses 60kv, 012 pulses 60kv, 012 pulses 60kv, 012 pulses</p>
<p>KNEE**</p> <p>AP Oblique Lateral</p>	<p>63kv, 022 pulses 60kv, 018 pulses 60kv, 018 pulses</p>
<p>OTHER</p> <p>Calcaneal Axial Harris Beath Sesamoids</p>	<p>60kv, 018 pulses 60kv, 022 pulses 60kv, 012 pulses</p>

SMALL (Child under 8): lower 2-4 pulses

LARGE: add 2-4 pulses

CAST: add 4 pulses

*All views to be shot at 15ma setting. *This is only a reference. kv and pulses can be adjusted when necessary.*

Calibration:

Luminance: 55kv, 008 pulses (or highest possible)

Positional: 45kv, 006 pulses (or lowest possible)

© 20/20 Imaging LLC - All Rights Reserved. *This chart is only a guide. 20/20 Imaging is not responsible for under/over exposure, or improper patient exposure problems. Your result may vary. For assistance, please call support at 866-734-6034. **Knee views are not supported and may be taken at your own risk.